


# SANIJET Pipeless™ Hydrotherapy System

## CLEANING AND DISINFECTION

Pure, clean and hygienic - SANIJET Pipeless™ hydrotherapy products have the only hydrotherapy system independently tested and certified by NSF International (formerly National Sanitation Foundation) for disinfection and complete hygiene.

### IMPORTANT! TOTAL DISINFECTION REQUIRES TWO DISTINCT STEPS

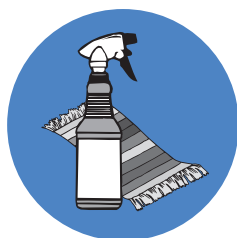

1. Cleaning:  
Remove residue with thorough and effective cleaning.



2. Disinfection:  
Apply an EPA registered disinfectant, covering 100% of all surface areas for the required contact time, to kill bacteria, viruses and fungi.



### Disinfectant Selection

SANIJET recommends the use of an EPA registered disinfectant with tuberculocidal, germicidal, pseudomonacidal, fungicidal and virucidal efficacy. DISPATCH® Hospital Cleaner Disinfectant with Bleach is a 2-in-1 solution providing quick and effective cleaning and disinfection.

### Recommended Cleaner / Disinfectant

DISPATCH® Hospital Cleaner Disinfectant with Bleach - EPA Reg. No. 56392-7; EST. No. 62801-MI-001. Kills bacteria and viruses in 1 minute and TB in 30 seconds. Available in 32 oz trigger spray and 64 oz refill bottle.



### Certification

- Meets NSF International P-182 Protocol for pipeless whirlpool bath disinfection.
- Using the above listed disinfection procedures with DISPATCH® Hospital Cleaner Disinfectant meets OSHA Directive CPL 2-2.69 regarding the protection of healthcare workers from the risk of contamination of blood borne pathogens.

# SANIJET Pipeless™ Hydrotherapy System

## DISINFECTION PROCEDURES

1

### Step 1: Preparation & Cleaning

- a. Drain the bathing well of all water.
- b. Rotate each jet counter clockwise and remove it from the jet case.
- c. Disassemble the jet assemblies (three pieces each) and place them in the sink and use a non-abrasive antibacterial cleaner to remove any visible residue - rinse with tap water.
- d. Use a non-abrasive antibacterial cleaner to clean the bathing vessel, jet cases, drain stopper and overflow mechanism – rinse with tap water.



2

### Step 2: Disinfection

- a. Spray the surface area of the jet assemblies and the bathing vessel until thoroughly coated with an EPA-registered disinfectant.
- b. Allow to set for the required disinfection contact period (refer to manufacturer's specifications) then rinse surfaces with tap water.



3

### Step 3: Reassembly

- a. Dry the jet assemblies and bathing vessel.
- b. Reassemble the jet assemblies, reinsert and rotate them in a clockwise direction to securely lock in place (locate logo at top of jet face for proper reassembly)



## Additional Notes

- SANIJET Pipeless™ Hydrotherapy system cleaning and disinfection takes approximately 5 minutes.
- NEVER use abrasive cleaners on any surface area.
- After each use, disinfect the system with the recommended disinfection procedures for maximum sanitation.
- Pipeless systems do not require a "fill and drain cycle" during disinfection, providing improved water conservation.
- For applications requiring quick cleaning between treatments, SANIJET recommends a second set of "clean" jet assemblies for quick replacement.